



ProsthStyle Prosthesis Cover

Available in different skin colors and sizes

APPLICATION

ProsthStyle Prosthesis Cover provides a durable and 3D realistic cosmetic finish for most types of leg prostheses. It is pulled over the cosmetic foam cover to provide a final protective covering for an endoskeletal or exoskeletal prosthesis. It may be cut off, tailored, glued to the prosthesis or worn over prosthetic socket directly on the skin. The material is dermatological impeccable.

DESCRIPTION

Highly elastic ProsthStyle Prosthesis Cover increase the durability of the foam cover and prosthetic components by providing a layer of protection against such environmental hazards as abrasion, soiling, staining, and deterioration from the effects of UV rays. It is made from very elastic and durable material with no risk of scrape.

APPLYING THE COVER

Pull on ProsthStyle Prosthesis Cover to the prosthesis, check the position of knee joint and mark appropriate length. If the Cover needs to be cut, pull it off. Protect the seam by priming it with universal glue and wait for the glue to dry. Cut the Cover with sharp scissors. Apply the Cover to the prosthesis.

If you want to fix the Cover on the prosthesis, use universal glue or double – sided tape. Glue instructions: The sockets proximal part should be sandpapered to get a rough surface, carefully remove all sandpaper dust. Apply the universal glue and follow its instructions. When the glue is dry cut off surplus material proximal to the brim.

NOTE

Do not bring the cover in contact with open flames since it is not flame-resistant.

CARE

ProsthStyle Prosthesis Cover can be washed in a washing machine at a temperature up to 40°C/105°F.

Do not tumble dry.

Do not bleach.

Do not iron.

Do not dry clean.



Prosthstyle Prosthesis Cover may be resistant to chemicals like alcohol, disinfectant, acetone. Before use the chemicals we recommend test on waste or unusable part to convince about not damaging ProsthStyle Prosthesis Cover.


ORDERING INFORMATION

Example:

PSPC – F – L – XL – 09

PSPC (ProsthStyle Prosthesis Cover) - (Male/Female) – (Left/Right) – (Size) – (Color)

Measurement chart (circumference in centimeters)

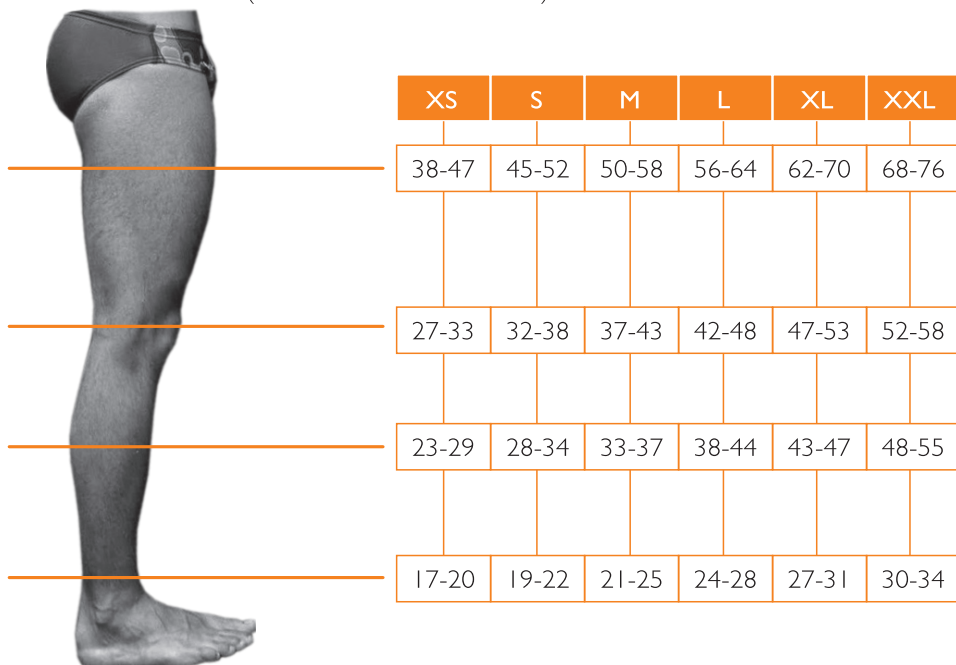

The length of ProsthStyle Prosthesis Cover is 100 cm.

WARRANTY

ART-LEG d.o.o. (ART-LEG) offers the first-time customer a 1-month warranty for this product, calculated from the date of delivery to the user. The warranty covers material defects and manufacturing faults. The warranty applies, if the products have been used properly and have not been provided with unapproved modifications. In addition, the products must be applied under responsibility of a person who must comply with all instructions and requirements listed in user manual. ART-LEG's only obligation regarding this product warranty is to repair the damaged product or to send free replacement.

Important!!! ART-LEG does not assume responsibility for the selection of skin colors, gender, side and size of the ordered product. The warranty does not cover replacing ordered product because of above reasons.

PACKING

1 pc. ProsthStyle Prosthesis Cover

ACCESSORIES

Color guide



Litostrojska cesta 44D
SI-100 Ljubljana, Slovenia
info@art-leg.com
www.art-leg.com